
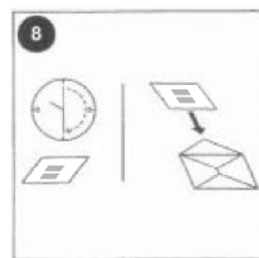
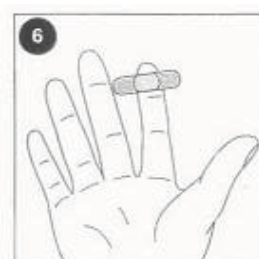
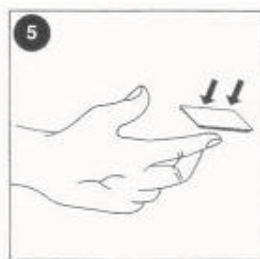
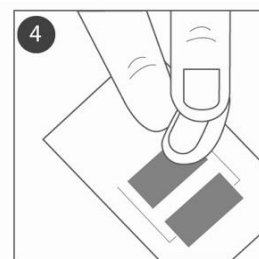
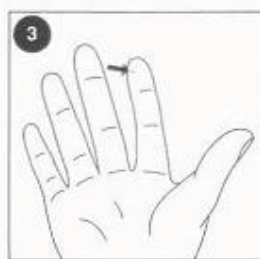
DRIED BLOOD SPOT COLLECTION PROCEDURES

Fingerstick Collection Instructions

1. Instruct the applicant wash his/her hands in warm, soapy water. Rinse and dry completely.
2. Clean site of skin puncture thoroughly with alcohol swab and allow finger to air dry. Remove filter paper from envelope.
3. With lancet, firmly puncture finger near tip but, slightly to the side. Ensuring a free flow of blood without excessive squeezing is important, as this may dilute the blood with tissue fluid.
4. Place the collection card on clean, dry surface. With applicant's arm slanting downward, palm upright, place drops of blood in box to left. Add blood drops to card until the strip turns red at the first (dark) line. **DO NOT** touch collection strip in the middle of the card.
5. Once the card is filled, then apply pressure to fingertip with alcohol swab until bleeding stops.
6. Place adhesive bandage over puncture site.
7. **DO NOT** attempt to re-cap lancet. Place used lancet in disposal tube, and dispose of as per your company's protocol. **NEVER RETURN A USED LANCET.**
8. Allow blood on filter paper to air dry for at least 20-30 minutes. Place completed barcode label where indicated on filter card. Place dried filter paper back into envelope. Return completed Dried Blood Spot (DBS) Envelope by placing in the outside pocket of the return shipping bag along with the urine collection kit, if sending the DBS card separate from the urine kit then place inside the return shipping bag and seal at top.

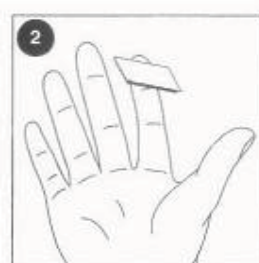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